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EN- Please read these safety instructions carefully before use, and keep them for future question.

1. Safety indications

- ♥ ALWAYS consult with your physician or supplier to determine proper adjustment and use of the device.
- ♥ Don't store outside and don't let under the sun (Beware : don't let the product in a car in case of high outside temperature !!).
- ♥ Don't use in extreme temperature (above 38°C or 100°F or below 0°C or 32°F).
- ♥ Check carefully all parts of this item (screws, tubes, tips, grips...).
- ♥ Check regularly if all screws are tightened.
- ♥ Use only as a shower chair and in proper way : don't use as a lever, don't stand up on the chair.
- ♥ ALWAYS observe the weight limit on the labeling of your product.
- ♥ No modification must be made on this device. Use only original parts to replace the broken or defect one and let this be done by the technical staff.
- ♥ After unfolding or assembling the chair, make sure that it is securely locked in OPEN position and level to the ground before use.
- ♥ DO NOT hang anything on the chair.
- ♥ ALWAYS test to see that the chair and attachments are properly and securely locked in place before use.
- ♥ In case you didn't use your chair for a long time, let it checked by the technical staff, before use.
- ♥ Do not store outdoor, the chair must be used at home.
- ♥ Check the chair before use, in case of doubt ask your physician or provider.
- ♥ Use only the seat on a flat and solid ground.
- ♥ Do not lean out of the chair to avoid losing balance.
- ♥ Any serious incident that has occurred in relation to the device shall be reported to the manufacturer and the competent authority of the Member State in which the user and/or the patient is established.

2. Intended use, indications and contraindications, installation

Intended use:

The Herdegen Shower Seat is designed to provide safety, comfort and independence for people who have difficulty standing in the shower.

The ergonomic seat ensures comfort for the user and has drainage holes for fast dispersion of water. It folds away when not in use.

Indications

Inability to get in/out of the bath/shower independently and to sit or stand in the bath/shower independently.

Functional impairment or lack of functional capability of the lower extremities
 Severe limited trunk control with enough residual functions of the upper extremities.

Contraindications

Don't use in case of:

- Cognitive disorder - Loss of balance - Sitting impairment - Inadequate hip and knee flexion and inability to transfer safely on / off bath seat independently.

Only specialists can adjust the product and teach you how to use it. Ask the technical staff how to use this product and make sure that you understood clearly their explanation.

In case of doubt, don't hesitate to ask again.

Beware ! The plastic shell seat has holes to drain water away from seating area. In very few cases, those drainage holes have the potential to entrap genitalia or rectal prolapse, leading to pain and serious injury such as cuts, lacerations and testicular bruising. In case of doubt, please consulte with your physician or supplier to determine a safe use of the device. You can also place material over bath seats to prevent flesh or anatomy passing through.

Installation

Remove Safety frame from packaging and check for cracks and damages. If you see any, or suspect a fault, do not assemble the product; contact your supplier immediately. Content of the box: 2 adjustable legs, 1 seatplate, 5 screws, 5 plain washer, see [A]

1- Slide 4 legs into the 2 frames by depressing push buttons and sliding them into the frames, follow step [B].

2- Insert the two frames into the slots in the seat and fasten them on the seat with the screws and plain washers , follow step [C].

3- Adjust the height by depressing the push button and sliding the telescopic leg to the desired position, follow step [D]. The push button automatically locks when it comes to the hole. Beware risk of injury! Make sure that push buttons "snap" correctly through holes and that the frame is stable. All legs must be adjusted at the same height. Check the stability before using the safety frame.

3. Cleaning

The shower chair has to be cleaned regularly with a mild cleaning agent and a soft cloth.

4. Materials

The product is made up of aluminium (doesn't rust) tubing, and various plastic components.

5. Disposal

Dispose of this product in accordance with your regional waste disposal regulations.

6. Technical specifications

This product has a 2 years warranty, from the date of purchase, against any manufacturing defect. Duration of use: 2 years.

This product is biocompatible. Test : EN-ISO 111 99-1.

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