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ENG Instructions for use

Edition: AD240624
Catalogue number: DRQM3A

• Versatile use in sports and medicine
• Perfect for rehabilitation and balance exercises
• Resistant to dynamic loads
The gym ball is designed for therapy to strengthen back muscles, increase joint mobility, improve fitness and motor coordination, in neurological disorders, as well as in the correction of postural defects.
The Qmed Anti-Burst Gym Ball is a safe device. The ball's material and ABS system prevent tearing and sudden loss of air even in the case of mechanical damage. The ball is designed for indoor use on a hard and flat surface.

Indications:
• For increasing joint mobility
• For posture correction exercises
• Mild back and neck pain that can be reduced as a result of physiotherapy and rehabilitation activities

Contraindications:
• before starting new sets of exercises, it is recommended to consult a physiotherapist.
• pain during exercise
• following dislocations or bone fractures
• incomplete bone fusion
• inflammation of joints and periarticular tissues
• oedemas
• fresh post-surgical scars or other wounds

Usage:
• Before inflating, remove the ball from the packaging and leave it at room temperature for 1-2 hours.
• Remove the plug and place the pump nozzle in the hole.
• Pump the ball gradually so that the material stretches slowly.
• After inflating, quickly remove the pump nozzle and insert the plug.
• The ball may only be used in a safe environment to prevent the user from getting hurt in the event of loss of balance. Therefore, make sure that there is no furniture with hard and sharp surfaces or other dangerous objects within the user's reach.

Note! Do not inflate beyond the maximum allowed size marked on the package. Measure the ball diameter. Keep the ball away from sharp objects, do not expose it to direct sunlight. The ball material may need to be expanded to reach the expected diameter. It is recommended to fill the ball with air to approx. 80-85% of its size and add more air after a few hours or so to obtain its full size.

Maintenance: After use, wash the product under running water with mild detergent. Dry away from artificial heat sources.

Diameter:
45 cm – for a height of 140 cm
55 cm – for a height of 155-169 cm
65 cm – for a height of 170-184 cm
75 cm – for a height of 185-200 cm
85 cm – over 200 cm

Composition: PVC
Intended user:
The medical device is intended for the general public (a non-professional user).
Disposal of the medical device: The device must be disposed of following the applicable local environmental regulations.

Caution: Any serious incident that occurs in connection with a device must be reported to the manufacturer and the competent authority of the Member State (applicable to the EU) where the user and/or patient is based.

Description of symbols used:

| | | | | | |
|--|---------------------|--|--|--|---|
| | Manufacturer | | Lot/batch number | | Medical device |
| | Date of manufacture | | Unique identification code of the product-type | | Compliant with Regulation (EU) 2017/745 |
| | Warning | | Unique catalogue number | | Refer to the instructions for use |

mdh Sp. z o.o.
ul. Maratońska 104
94-007 Łódź, Polska (EU)
tel. +48 42 674 83 33
e-mail: port@meyragroup.com

PELOTA DE REHABILITACIÓN CON SISTEMA ABS CON BOMBA REHABILITATIONSBALL MIT ABS-SYSTEM UND PUMPE

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